Owner's Guide

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
Manual del propietario

Soft Touch Collector Cup

Coupe collectrice Soft Touch

Concha recolectora Soft Touch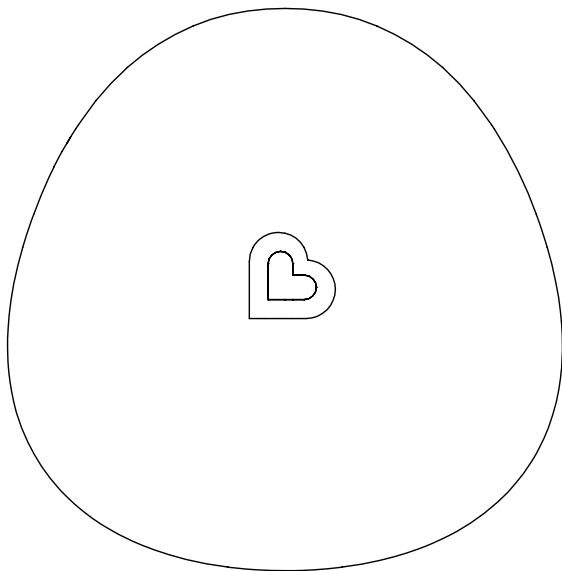

KEEP FOR FUTURE REFERENCE

Please read the instructions carefully before using this product.

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
IMPORTANT SAFEGUARDS

WARNING:


Wash and inspect before use. Do not use abrasive cleaners or cleaning products. Allow product to completely air dry before using. Top rack dishwasher safe. Do not store near any sharp objects. Store product in a cool, dry place and avoid direct sunlight. Use product for temporary storage only. Milk stored in product for too long should not be fed to child. Do not microwave the breast milk. Discard and replace at first sign of damage. This product is not a toy, adult use only.

INCLUDED PARTS

Please check that all parts are included before you begin. If any parts are missing or damaged, please contact our Customer Satisfaction Team at **1-800-344-2229**.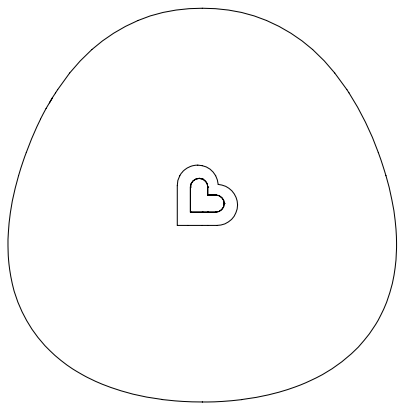

Collector Cup

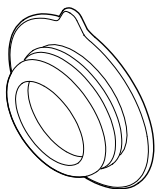Stopper

BEFORE FIRST USE

- Place the collector cup and stopper in boiling water for 5 minutes.
- Let it cool down for 10 minutes.
- Air dry completely.

AFTER EACH USE

- Separate the parts. Wash all the parts immediately after each use in warm, soapy water. Allow to air dry completely.

NOTE: Only use soft bristle brushes or soft sponges when cleaning the product. Hard scourers may scratch the surface.


- Top rack dishwasher safe.
- Microwave steam sterilize safe.

IMPORTANT: Do not microwave breast milk.


- Avoid contact with sharp objects when storing.

HOW TO USE

1. Place the collector cup onto your breast, with your nipple in the center of the opening.

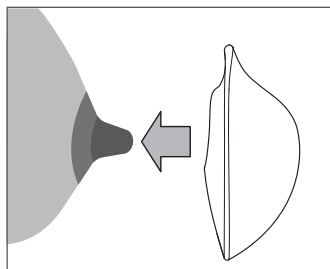
2. Gently press the collector cup against your breast to remove air and to create a gentle seal. If needed, adjust the position for comfort or for better suction.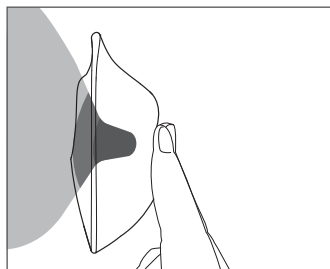


3. Pull your nursing bra over the collector cup. If you feel the cup loosen, press the cup again to create more suction. Reapply pressure during letdown, if needed.


NOTE: Avoid sudden movement as it may loosen the suction.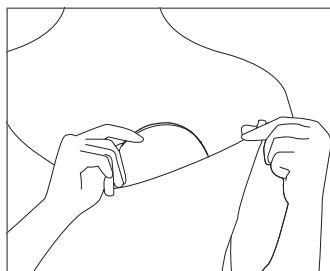

4. Check the milk level after letdown begins. Lean forward slightly then remove the collector cup when it reaches the maximum fill level to avoid accidental spilling. When you're finished, release the seal around the opening by inserting your finger.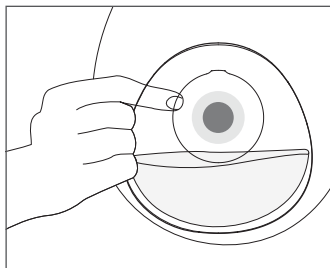

5. Pour the collected breastmilk into a bottle or keep it in the collector cup for temporary storage. Insert the stopper into the collector cup to prevent contamination. Make sure the stopper is inserted tightly to prevent milk from leaking. Store the collector cup on domed surface with heart logo facing down.

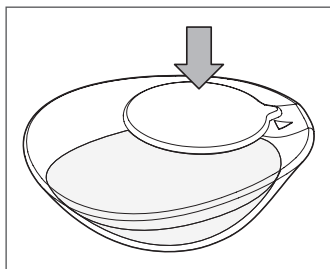NOTE: Lid should only be used for temporary storage in room temperature or in refrigerator.

CONTACT US

If you have any questions or wish to provide comments about this product, feel free to contact our Customer Satisfaction team at **1-800-344-BABY** (2229) (M-TH 8:00am- 5:00pm PST, F 8:00am-1:00pm PST) or visit our website at **www.munchkin.com**.

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