Owner's Guide

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
Manual del propietario

Silicone Breast Pump

Tire-lait en silicone

Extractor de leche de silicón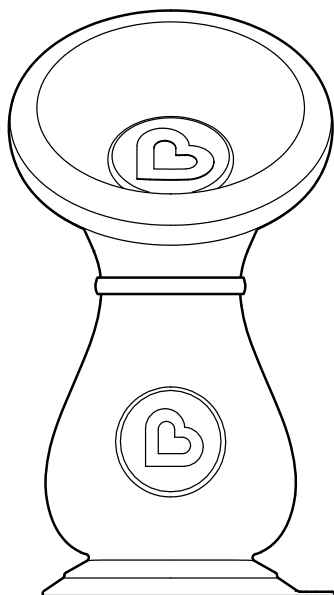

KEEP FOR FUTURE REFERENCE

Please read the instructions carefully before using this product.

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
IMPORTANT SAFEGUARDS

WARNING:


Wash and inspect before use. Do not use abrasive cleaners or cleaning products. Allow product to completely air dry before using. Top rack dishwasher safe. Do not store near any sharp objects. Store product in a cool, dry place and avoid direct sunlight. Use product for temporary storage only. Milk stored in product for too long should not be fed to child. Do not microwave the breast milk. Discard and replace at first sign of damage. This product is not a toy, adult use only. The lanyard is solely designed and intended to be worn by adult. Do not place the lanyard around the child's neck.

INCLUDED PARTS


Please check that all parts are included before you begin. If any parts are missing or damaged, please contact our Customer Satisfaction Team at **1-800-344-2229**.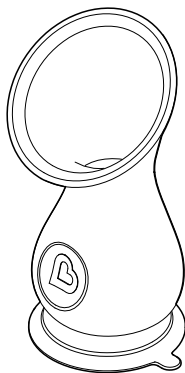

Breastpump

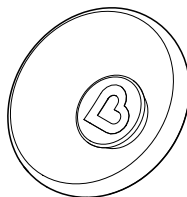Lid

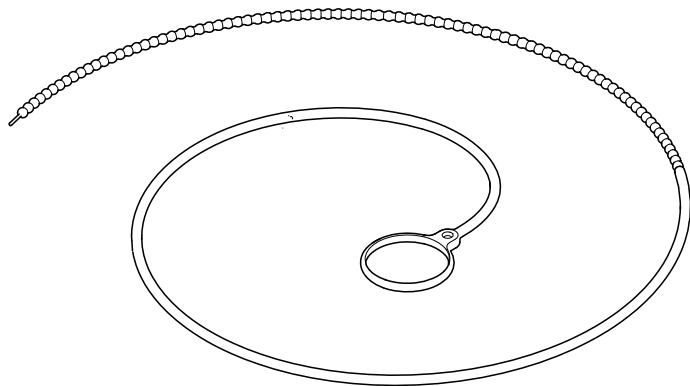Lanyard

BEFORE FIRST USE

- Remove the lanyard from the breastpump before cleaning.
- Place the breastpump and lid in boiling water for 5 minutes.
- Let it cool down for 10 minutes.
- Air dry completely.

AFTER EACH USE

- Separate the parts. Wash all the parts immediately after each use in warm, soapy water. Allow to air dry completely.

NOTE: Only use soft bristle brushes or soft sponges when cleaning the product. Hard scourers may scratch the surface.


- Top rack dishwasher safe.
- Microwave steam sterilize safe.

IMPORTANT: Do not microwave breast milk.


- Avoid contact with sharp objects when storing.

HOW TO USE

1. Place the small loop of the lanyard around the breastpump. Then put the lanyard around your neck to prevent spills during use. Adjust the length as needed.

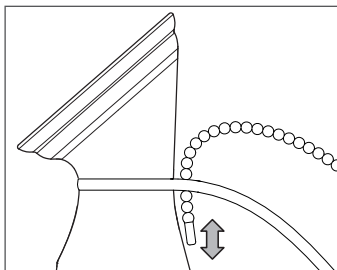
2. Center the opening of the breastpump around your nipple.
3. Squeeze the breastpump to remove air and create suction. If needed, adjust the position for comfort or for better suction. Keep in mind that letdown might take up to 90 seconds to begin.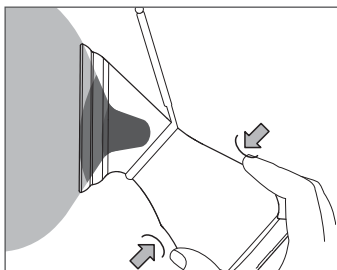


TIP: To get even more suction from your pump, fold back the flange to expose the neck of the pump. Center the opening over your nipple and squeeze the pump to generate suction.

4. When the pump reaches the maximum indicator level, gently squeeze the mouth of the breastpump to release the seal. Hold the base to prevent spilling.

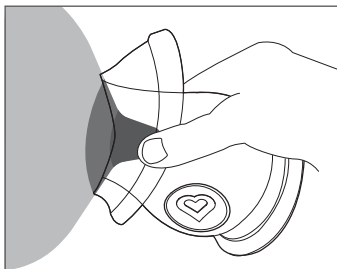
5. Place the pump on the table and press down slightly to activate the suction base. This will secure it firmly and prevent falls. To move it, gently lift the release tab on the suction base while holding the pump.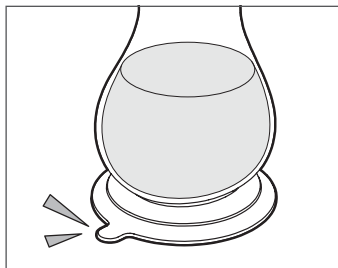

6. Pour the collected breast milk into storage or keep it in the pump for temporary storage. Place the lid over the mouth of your breastpump to prevent contamination. Make sure the lid is firmly inserted to prevent milk from leaking.

NOTE: Lid should only be used for temporary storage in room temperature or in refrigerator.

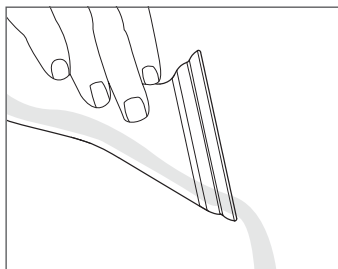CONTACT US

If you have any questions or wish to provide comments about this product, feel free to contact our Customer Satisfaction team at **1-800-344-BABY** (2229) (M-TH 8:00am- 5:00pm PST, F 8:00am-1:00pm PST) or visit our website at **www.munchkin.com**.

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© 2024 MUNCHKIN, INC., A WHY BRANDS COMPANY. 7835 GLORIA AVE., LOS ANGELES, CA 91406.
MUNCHKIN BABY CANADA LTD., 3-8460 MOUNT PLEASANT WAY, MILTON, ON L9T 8W7. MADE IN